EMOS Technology GmbH
 Gewerbestrasse 10
 D- 88636 Illmensee
www.emostechnology.com
Info@emostechnology.com

EMOS Technology GmbH
 Gewerbestrasse 10
 D- 88636 Illmensee
www.emostechnology.com
Info@emostechnology.com

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Instruction for use

1. Table of contents

1. Tabel of contents.....	1
2. Symbols	2
3. Intended use.....	3
4. Foreseeable misuse.....	4
5. Safety instructions	4
6. Scope of delifery / combinations	4
6.1 Scope of delivery.....	4
6.2 Combinations	5
6.3 Assembly / Disassembly	5
7. Cleaning and care	5
8. Packaging	6
9. Storage	6
10. Repair / Service.....	6
11. Functional description of the Mini PortLed	6
12. Standards / Directives	7
13. Disposal	7

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Thank you for the confidence you have demonstrated by purchasing this device.

In order to make the best use of it while having all the necessary precautions at your disposal, it is essential for you to become acquainted with this instruction for use.

The sentences displaying the symbol  correspond to points requiring special attention.

The sentences displaying the symbol  represent items of information.

This user manual is an integral part of the device. It must be made available to the user. Proper use and correct handling of the device entail following these instructions. The user alone shall be responsible for any damage that may result from improper use.

Guarantee:

We provide a complete guarantee in case of manufacturing or quality defects. In case of visible defects which are caused by production failures or the use of defective materials, the products are reworked or replaced free of charge.

Not included in the guarantee are all damages caused by improper usage such as mechanical impact, downfall, overload etc. The warranty becomes invalid for repairs done by unauthorized persons.

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Artikelnummer

Target group

This manual is intended exclusively for trained and qualified personnel.

Mini PortLed are to be used exclusively by qualified personnel for clinical purposes.

EMOS Technology recommended to check the suitability of the product for the intended procedure prior to use.

2. Symbols

2.1 Warnings

The following signal words serve to distinguish between warning notes according to the type of danger:

- ➔ Note warns about damage to property.
- ➔ Caution warns about a slight physical injury.
- ➔ Warning warns about a serious physical injury.
- ➔ Danger warns about danger to life.

Structure of warning notes:

SIGNAL WORDS!

Type and source of danger!

- ➔ Measure required to prevent danger.

2.2 Other symbols

Notes

- ① Note on appropriate use of endoscopes and accessories.

Instructions

Structure of instructions:

- ➔ Instructions on an action to be carried out. Information on result if necessary.

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-  Keep away from sunlight
-  Keep in a dry place
-  CE-marking
-  Electronic and electrical equipment put on the market after 13/08/2015. This symbol indicates that this product must not be processed with household waste.

3. Intended use

Main function:

The Mini PortLed from EMOS Technology GmbH is intended for the generation of light during endoscopic diagnostic interventions. It corresponds to the Regulation (EU) 217/745.class I.

Duration of use:

Temporary use (<60 minutes under normal conditions).
For longer investigations, EMOS recommends using a compatible cold light source.

Medical indication:

Standard endoscopy examination in sinus copy, otoscopy and laryngoscopy.

User profile:

Generally trained specialist for medical endoscopy. Specialist in ENT medicine.

Patient group:

Patients of all ages with no restrictions who require standard endoscopy.

Application organ:

Natural body openings for medical endoscopic examination of the throat / nose / ears.

Application environment:

Clinics and medical practices for specialists in endoscopy, mainly in the field of ENT. Operation and use may only be carried out by trained and instructed personnel.

Contraindication:

Not outside of a medical environment, not in potentially explosive areas.

Sterility:

Not sterile.

Reprocessing:

Not a disposable product, cleaning of the Mini PortLed by wiping disinfection. See list of disinfectants and procedures tested and recognized by the Robert Koch Institute (RKI).

<https://www.rki.de>

⚠ Note the concentration and exposure time of the respective disinfectant manufacturer

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Special Features

- Greater freedom of movement thanks to wireless use of the endoscopes
- Intensive Lightning
- 4 adjustable brightness levels plus Stroboscope mode
- Water resistant (IPX8)
- Ergonomic design
- Long lifetime battery
- light weight

Working Environment

- Ambient temperature: -40 ° C to + 70 ° C
- Humidity: 10% to 100%, including condensation
- Air pressure: 50 kPa to 106 kPa

4 Foreseeable misuse

- ☞ Do not use products for other applications

5 Safety Instructions

Follow these warnings and precautions when handling this Mini PortLed.

- ⚠ Read User Manual.
- ⚠ This user manual is an integral part of the apparatus. It must be made available to the user. Proper use and correct handling of the apparatus entail following these instructions. The user alone shall be responsible for any damage that may result from improper use.
- ⚠ CAUTION!
Ignoring the instructions can lead to severe injury.
- ⚠ WARNING!
If Creutzfeld-Jacob syndrome is suspected or diagnosed, EMOS Technology GmbH recommends not to reuse the Mini PortLed, but to dispose of it.
- ⚠ WARNING!
If a malfunction occurs during use on the patient, stop using the device immediately.
- ⚠ CAUTION!
Never autoclave or ultrasonic cleaning the Mini PortLed (maximum temperature 70 ° C).
- ☞ Comply with the conditions of use and storage
- ☞ The apparatus should be opened only by a competent technician authorized by the manufacturer
- ☞ Use only the accessories supplied with the apparatus or recommended as options by the manufacturer
- ☞ This apparatus is not a flame-proof device. Therefore, it must not be used in the presence of flammable gases
- ☞ Do not place any heavy objects on it.

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- The surface temperature of the endoscope connection can reach 41°C (after 60 minutes' use). Therefore, avoid any contact with this area
- Prior to activation, ensure that neither the Mini PortLed nor the endoscope are damaged. Deteriorated cables and connectors must be replaced immediately. Perform a functional test before commissioning.
- Do not drop the product. If the product falls, do not reconnect the product but send it back to your authorized distributor or directly to the EMOS Technology GmbH after-sales service department
- Never look at the light output or the distal end of the endoscope.
- ⚠ Any modification of this device without authorization from the manufacturer is prohibited. If the device is modified, an inspection and a test must be carried out to ensure that the device complies with the safety regulations.
- ⚠ Apparatuses connecting to the input/output ports must comply with the IEC Standard
- ⚠ After use, be sure to carefully unscrew the Mini PortLed.

6 Scope of delivery / combinations

6.1 Scope of delivery

- Scopes of delivery see packaging.

6.2 Combinations

- ① For combinations with other products, see overview table
- Combine products and accessories with other devices if the intended use in the instruction manual or the operating instructions allows it.
- The technical data in the instruction manual or the operating instructions allows it.

The Mini PortLed can be connected to the following items:
Endoscopes with ACMI and Wolf adapter.

6.4 Assembly / Disassembly

The endoscope is ready with the Mini PortLed.

Carefully put the Mini PortLed onto the thread of the connector for light guide (ACMI and Wolf adapter).

- By turning the Mini PortLed clockwise without any tools until it is firmly seated, the Mini PortLed is firmly screwed to the endoscope.
- The thread may only be oiled with a physiologically harmless (medically approved oil) according to DAB 10.
- Disassembly is carried out by turning the Mini PortLed counterclockwise.

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7 Cleaning and care

- ☞ You may clean the external tube of the endoscope with a dry cloth
- ☞ In case of dirt on the outside of the glass pieces use a cotton-wool-tip moistened with acetone or alcohol for cleaning.

8 Packaging

- ☞ The transport packaging is not intended for sterilization and storage. Therefore, do not store products in the transport packaging.
- ☞ The products must be completely dry before storage.
- ☞ Only store and pack products individually.
- ☞ Store products in a dry, clean, dust-free and well-ventilated environment and in a protected place at room temperature (free of corrosive vapors). Avoid temperature fluctuations.

9 Storage

- ☞ Store the products in a dry, clean and dust-free environment at moderate temperatures (free from corrosive vapors).

10 Repair / Service

To guarantee the operational safety of the products:

- ☞ Let exclusively the respective dealer or a repair facility authorized by EMOS TECHNOLOGY GMBH carry out repairs works.
- ☞ Use exclusively original spare parts for repair works.

Prior to return

- ☞ Clean and disinfect products and accessories.

11 Functional description of the Mini PortLed

On / Off: Single click on the side switch to switch the light on / off.

Changing the brightness: if the light source is switched on, press and hold the side switch for 1 second to change the brightness level (brightness level 1-3). The mode is selected by releasing the side switch.

Lowest brightness level: if the light source is switched off, press and hold the switch for longer than 1 second.

Power Mode: press the side button quickly twice to switch on the highest brightness level.

LOCK: when the Mini PortLed is off, press and hold the switch for a few seconds. The lowest brightness level is activated briefly, then the Mini PortLed switches off again and the transport lock is activated.

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This means that the Mini PortLed no longer reacts to accidental pressure on the side switch. It is now secured against unintentional switching on, this is signaled by a red LED in the side switch for approx. 3 seconds.

UNLOCK:

To deactivate the lock, press and hold the side switch for about three seconds.

CHARGING THE BATTERY

Please charge the battery before using it for the first time. Use the supplied USB charger and connect it with the USB charging cable and the hand lamp. The charging indicator lamp lights up red during charging and changes to green when charging is complete.

BATTERY CHARGE

The remaining battery charge is indicated by an LED in the switch (red <10%, yellow 10 ~ 60%, green > 60%).

12 Standards/Directives

This appliance was designed and made by a company with a certified quality system. It meets the requirements of the electrical safety standards and Electromagnetic Compatibility ad hoc. (93/42/EWG; 2007/47/EG; EN 55015; EN 61547; EN ISO 13485).

All data below are test results according to ANSI NEMA FL 1-2009 standards.

13 Capability characteristics

Illuminance

Power Mode: 160.000 lux

Level 1: 88.000 lux

Level 2: 12.000 lux

Level 3: 1.700 lux

Strobo-Mode: 88.000 lux / 13 Hz

Waterproof IPX8

Length: 7,5cm

Battery charging time: 3Std

Battery

3,7V

550mAh

2.0Wh

14 Disposal

⚠ Thoroughly clean and disinfect products before disposal.

⚠ Dispose of packaging and used parts in accordance with country-specific regulations.

This unit bears the recycling symbol in accordance with directive 2002/96/EEC concerning Waste Electrical and Electronic Equipment (WEEE or WEEE). By disposing of this device properly you will help to prevent any harmful consequences to the environment and to human health. The symbol  on the appliance or the accompanying documents indicates that this product may never be treated as household refuse. It must, consequently, be taken to a refuse collection center charged with recycling electrical and electronic equipment.

For disposal, observe the standards relating to the dumping of refuse in force in the country of installation.

For more details about dealing with, recovering and recycling this product, please contact your nearest dealer who will tell you what to do.

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The Mini PortLed are reusable products. The lifespan depends on the frequency of use as well as care and careful handling